



Purpose

This document provides guidance on how to compare prior image sessions using Optos*Advance* v24.

NOTE: Full operating instructions, device warnings and cautionary messaging are provided in the **Help** section of Optos*Advance*. Once logged into the software, select the **Help** icon from the toolbar to search or browse guidance topics. To confirm your version of the software, hover over the Optos*Advance* logo.



Contents

Purpose	1
Timeline Visibility	1
Viewing Prior Studies	
Scrolling Through Priors	
Comparing More than Two Visits	

Timeline Visibility

When prior studies exist, a timeline will display under the toolbar, as shown below. If the timeline does not appear, but priors do exist, it is collapsed. Follow the step below to make the timeline visible:



1 | Select the blue down arrow button in the upper right corner below the OptosAdvance logo.







Viewing Prior Studies

If there are prior studies for the patient, you can compare today's image modality with the most recent prior image modality side-by-side by doing one of the following:

- Select the **Prev. and Next** icons on the toolbar.
- Press the next arrow on your keyboard.
- Manually activate the stage by selecting the Hang. Prot.
 (Hanging Protocols) icon to show the menu and choose the desired stage.
 - Prior stages are marked by an asterisk (*) after the laterality and image modality.



Scrolling Through Priors

Once a prior hanging protocol is active in the viewports, you can scroll through additional prior images side-by-side.

NOTE: Three or more prior sessions are required for this functionality.

1 | Ensure your timeline displays **HP Priors Scrolling**.



- 2 | If **HP Priors Scrolling** is missing:
 - a. Select the **Settings** icon.
 - Navigate to System>Timeline and ensure Show HP Prior Navigation Panel is active.
 - c. Select **Save**. Return to **Directory**.







- 3 | Open your desired study and confirm **HP Priors Scrolling** is displayed in the timeline as shown in Step 1.
- 4 | Select a prior stage (indicated with an asterisk) to display a current image modality and the prior image modality.



- 5 | Select the **Forward** button on **HP Priors Scrolling** to change the right viewport to scroll through all prior matching modalities.
- 6 | Take notice of the blue numbered circles in the timeline. The viewports will display the matching blue circle in the bottom right corner of the viewport to indicate which date is shown.
- 7 | This functionality applies to all image modalities and OCT Analysis reports if the correct prior hanging protocol is selected.









Comparing More than Two Visits

To view more than two image modalities on the screen, follow the steps outlined below.

- 1 | Open the desired image modality to compare.
- 2 | Select **Study Layout** from the toolbar. Change your **Symmetric Layout** to the desired number of images you want to view.

For example, if you would like to see three images on the screen, change the study layout to 1x3.


NOTE: Layouts that contain more than 9 images may result in some titles hidden.




- 3 | Click the **All** icon at the top right of the screen to display the **Extended Patient Timeline**.
- 4 | Drag and drop any of the thumbnail images into a viewport to display the image.
- 5 | Change Laterality from the **OD** or **OS** icon on the toolbar.



6 | Change **Optomap Views** or **OCT Views** from the **View** icon on the toolbar as needed.

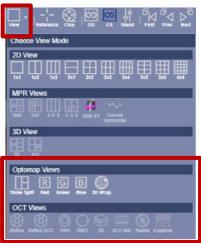

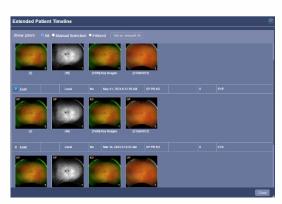



Optos UK/Europe +44 (0)1383 843350 ics@optos.com

Optos North America 800 854 3039 usinfo@optos.com Optos DACH
DE: 0800 72 36 805
AT: 0800 24 48 86
CH: 0800 55 87 39
ics@optos.com

Optos Australia +61 8 8444 6500 auinfo@optos.com



